



QIAstat-Dx RP Control Panel M360

INTENDED USE:

QIAstat-Dx RP Control Panel M360 is intended for use as an external positive and negative quality control to monitor the performance of *in vitro* laboratory nucleic acid testing procedures for the qualitative detection of bacterial and viral pathogens: Adenovirus, Bocavirus, Coronavirus 229E, Coronavirus HKU1, Coronavirus NL63, Coronavirus OC43, Human Metapneumovirus A+B, Influenza A, Influenza A H1, Influenza A H1N1 pdm09, Influenza A H3, Influenza B, Parainfluenza virus 1, Parainfluenza virus 2, Parainfluenza virus 3, Parainfluenza virus 4, Respiratory Syncytial Virus A+B, Rhinovirus/Enterovirus, *Bordetella pertussis*, *Chlamydomphila pneumoniae*, *Legionella pneumophila* and *Mycoplasma pneumoniae* on the QIAstat-Dx[®] Respiratory Panel performed on QIAstat-Dx systems. QIAstat-Dx RP Control Panel M360 is composed of synthetic RNA specifically designed for and intended to be used solely with the QIAstat-Dx Respiratory Panel.

PRODUCT SUMMARY and PRINCIPLE:

QIAstat-Dx RP Control Panel M360 is composed of 3 controls, QIAstat-Dx RP Positive A, QIAstat-Dx RP Positive B and QIAstat-Dx RP Negative. QIAstat-Dx RP Positive A and QIAstat-Dx RP Positive B contains surrogate control material composed of synthetic nucleic acid corresponding to genome segments of pathogens listed in Table 1 and 2. QIAstat-Dx RP Negative contains no nucleic acid.

Routine use of quality controls that are consistent lot to lot assists the laboratory in identifying shifts, trends, and increased frequency of random errors caused by variations in the test system, such as failing reagents. Early investigation can prevent failed assay runs.

COMPOSITION:

The QIAstat-Dx RP Control Panel M360 is comprised of 12 tubes, 4 tubes of QIAstat-Dx RP Positive A, 4 tubes of QIAstat-Dx RP Positive B and 4 tubes of QIAstat-Dx RP Negative, 300µL each. QIAstat-Dx RP Positive A and QIAstat-Dx RP Positive B contain synthetic RNA suspended in a non-infectious solution of buffers, preservatives and stabilizers. QIAstat-Dx RP Negative contains buffers and preservatives. Table 1 and 2 lists the pathogens that are monitored by the QIAstat-Dx RP Control Panel M360 when tested by the QIAstat-Dx Respiratory Panel on the QIAstat-Dx systems.

INSTRUCTIONS FOR USE:

1. Allow the control to be tested to come completely to room temperature (18° – 25°C).
2. Use the control as provided. DO NOT DILUTE.
3. Prepare a QIAstat-Dx Respiratory Panel Cartridge according to the QIAstat-Dx Respiratory Panel Instructions for Use.
4. Open the sample lid of the main port on the front of the QIAstat-Dx Respiratory Panel Cartridge according to the QIAstat-Dx Respiratory Panel Instructions for Use.
5. Immediately before use, mix the control thoroughly by inverting 3-5 times and tap the tube several times on the bench to remove any control caught in the cap before opening the tube.
6. Use the supplied transfer pipette to draw up the liquid control to the third fill line on the pipette (300µL).
7. Carefully transfer 300µL of sample volume into the main port on the QIAstat-Dx Respiratory Panel cartridge. Firmly close the sample lid of the main port until it clicks.
8. Visually confirm that the sample has been loaded by checking the sample inspection window in the cartridge.
9. The controls are single use only. Discard control tube after use according to your local and federal regulations.
10. Continue to process the control according to QIAstat-Dx Respiratory Panel Instructions for Use.

PRECAUTIONS, WARNINGS and LIMITATIONS:

- Do not dilute. Use the control as provided.
- This product is intended for *in vitro* analytical testing and is provided for Research Use Only, not for use in diagnostic procedures.
- This product is only for use with QIAstat-Dx Respiratory Panel on the QIAstat-Dx systems. It does not contain the entire genome of the respiratory pathogens listed in Table 1 and 2.
- This product is not intended for use as a substitute for the internal controls provided in the QIAstat-Dx Respiratory Panel.
- This product is slightly cloudy in appearance.
- This product does not contain any biological material of human or animal origin. Universal Precautions are NOT required when handling this product.
- Quality control materials should be used in accordance with local, state, federal regulations and accreditation requirements.
- QIAstat-Dx RP Control Panel M360 cannot be cloned, sold, or transferred without the explicit written consent of MMQCI.

STORAGE and STABILITY:

QIAstat-Dx RP Control Panel M360 should be stored frozen (-20°C or colder). Unopened QIAstat-Dx RP Control Panel M360 material is stable through the expiration date printed on the kit label when stored frozen. QIAstat-Dx RP Positive A and QIAstat-Dx RP Positive B and QIAstat-Dx RP Negative are for single use only. Discard after use according to your local and federal regulations.

EXPECTED VALUES:

Table 1 and Table 2 list the expected results when the controls are analyzed by QIAstat-Dx Respiratory Panel. Note that the reported Ct/EP may vary run to run, and among different instruments and laboratories.

The laboratory should follow Good Laboratory Practice (GLP) and establish its own performance characteristics for QIAstat-Dx RP Control Panel M360 in demonstrating adequate system performance.

Table 1: US customers Expected Results for QIAstat-Dx RP Control Panel M360

| Result Details | Positive A | Positive B | Negative |
|----------------------------------|--------------|--------------|--------------|
| Viruses | | | |
| Adenovirus | + Detected | Not Detected | Not Detected |
| Coronavirus 229E | + Detected | Not Detected | Not Detected |
| Coronavirus HKU1 | Not Detected | + Detected | Not Detected |
| Coronavirus NL63 | + Detected | Not Detected | Not Detected |
| Coronavirus OC43 | Not Detected | + Detected | Not Detected |
| Human Metapneumovirus A+B | + Detected | Not Detected | Not Detected |
| Influenza A H1 | + Detected | Not Detected | Not Detected |
| Influenza A H1N1 pdm09 | + Detected | Not Detected | Not Detected |
| Influenza A H3 | Not Detected | + Detected | Not Detected |
| Influenza B | Not Detected | + Detected | Not Detected |
| Parainfluenza virus 1 | + Detected | Not Detected | Not Detected |
| Parainfluenza virus 2 | Not Detected | + Detected | Not Detected |
| Parainfluenza virus 3 | + Detected | Not Detected | Not Detected |
| Parainfluenza virus 4 | Not Detected | + Detected | Not Detected |
| Respiratory Syncytial Virus A+B | Not Detected | + Detected | Not Detected |
| Rhinovirus/Enterovirus | + Detected | Not Detected | Not Detected |
| Bacteria | | | |
| <i>Bordetella pertussis</i> | Not Detected | + Detected | Not Detected |
| <i>Chlamydomphila pneumoniae</i> | Not Detected | + Detected | Not Detected |
| <i>Mycoplasma pneumoniae</i> | + Detected | Not Detected | Not Detected |

Table 2: NON-US customers Expected Results for QIAstat-Dx RP Control Panel M360

| Result Details | Positive A | Positive B | Negative |
|---------------------------------|--------------|--------------|--------------|
| Viruses | | | |
| Adenovirus | + Detected | Not Detected | Not Detected |
| Bocavirus | Not Detected | + Detected | Not Detected |
| Coronavirus 229E | + Detected | Not Detected | Not Detected |
| Coronavirus HKU1 | Not Detected | + Detected | Not Detected |
| Coronavirus NL63 | + Detected | Not Detected | Not Detected |
| Coronavirus OC43 | Not Detected | + Detected | Not Detected |
| Human Metapneumovirus A+B | + Detected | Not Detected | Not Detected |
| Influenza A | + Detected | + Detected | Not Detected |
| Influenza A H1 | + Detected | Not Detected | Not Detected |
| Influenza A H1N1 pdm09 | + Detected | Not Detected | Not Detected |
| Influenza A H3 | Not Detected | + Detected | Not Detected |
| Influenza B | Not Detected | + Detected | Not Detected |
| Parainfluenza virus 1 | + Detected | Not Detected | Not Detected |
| Parainfluenza virus 2 | Not Detected | + Detected | Not Detected |
| Parainfluenza virus 3 | + Detected | Not Detected | Not Detected |
| Parainfluenza virus 4 | Not Detected | + Detected | Not Detected |
| Respiratory Syncytial Virus A+B | Not Detected | + Detected | Not Detected |
| Rhinovirus/Enterovirus | + Detected | Not Detected | Not Detected |
| Bacteria | | | |
| <i>Bordetella pertussis</i> | Not Detected | + Detected | Not Detected |
| <i>Legionella pneumophila</i> | + Detected | Not Detected | Not Detected |
| <i>Mycoplasma pneumoniae</i> | + Detected | Not Detected | Not Detected |

ORDERING INFORMATION:

QIAstat-Dx RP Control Panel M360

Part Number: M360

Kit Contains: 12 tubes x 300µL

4 each Positive A, Positive B and Negative