**TECHNICAL NOTE # 12-002**

RE: INTROL® CF Panel II Control (p/n: G110)  
ASSAY: Luminex xTAG® Cystic Fibrosis 39 Kit v2 (CE-IVD)  
DATE: 12/15/17

Below is a table indicating the expected calls for the INTROL® CF Panel II Control on the Luminex xTAG® Cystic Fibrosis 39 Kit v2 (CE-IVD).

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1. A No Call for both dI507 and dF508 in Bottle a may occur due to the presence of I507V and F508C, which prevent binding of dI507 and dF508 WT probes.
2. Genotype call for S549R (T>G) in Bottle a may be HET or Mu D. Actual genotype is HET for S549R (T>G). The Mu D may occur due to the presence of the Q552X mutation (not detected by xTAG), which prevents binding of S549R (T>G) WT probe on the wild type allele.
3. Genotype call for 5T/7T/9T in Bottle a may be 7T or 7T/9T. Actual genotype is 7T/11T; however, 11T is not detected by xTAG. The 7T/9T call may occur due to possible 11T non-specific primer extension with the 9T probe resulting in a 7T/9T call.
4. Genotype call for S549N in Bottle b may be HET or Mu D. Actual genotype is HET (S549N/wt). The Mu D call may occur due to the presence of G551D mutant, which prevents binding of S549N WT probe on the wild type allele.
5. A Mu D call for dI507 in Bottle c is expected due to the presence of I506V, which prevents binding of dI507 WT probe.
6. A NS call for dF508 in Bottle c is expected due to the presence of I506V, which prevents binding of dF508 WT probe.

Please call us at 207-885-1072 x1 if you have any questions.

Sincerely,  
Maine Molecular Quality Controls, Inc.  
www.mmqci.com